

How do I clean my device?

Before cleaning your speech-generating device, please shut it down and unplug all cables. Keep the protective case on the device. The device should be cleaned using a disinfecting or hospital grade cleaning wipe for infection control. The cleaning should be done using a wipe style product only. If you do not have disinfecting wipes, you can spray a soft cloth with disinfecting spray use the cloth to wipe down the device. Wipe only outer surfaces of the device and allow the product to dry naturally to follow the wipe instructions. Do spray products directly onto the device, as they may saturate the device or allow moisture into the unit. Do not let moisture seep into the charging ports.

<https://www.tobiidynavox.com/en-us/support-training/faq-from-salesforce/how-do-i-clean-my-device/>

<https://saltillo.com/blog/post/tips-for-cleaning-disinfecting-your-prc-saltillo-device>